MENICARE PURE

MULTIPURPOSE SOLUTION

en

Instructions for use for *MeniCare Pure*

Multipurpose Solution for all rigid gas permeable contact lenses
This Instructions For Use contains important information.
Please read carefully and retain for future use.

INDICATIONS

- DISINFECTING: Effective after 30minutes
- CLEANING
- RINSING
- STORAGE

TO OPEN

The dropper tip is sealed. To open, screw the cap clockwise as far as it will go. The spike inside the cap will pierce the seal.

DIRECTIONS

Wash your hands thoroughly before handling your contact lenses:

- Rub the lens carefully with several drops of *MeniCare Pure* for 20 seconds.
- Place the lenses in the right and left lens holder and rinse them with *MeniCare Pure*.
- Fill the lens case with *MeniCare Pure* and soak the lenses for at least 30 minutes or overnight.
- Before insertion, rinse the lenses with *MeniCare Pure*.
- Keep the lens case clean and dry after use.

Lenses may be stored in the unopened case until ready to wear, up to a maximum of 30 days. If you store your lenses for longer periods of time, they must be cleaned and disinfected with *MeniCare Pure* solution every 30 days.

WARNINGS

- Never use tap water or non-sterile water with contact lenses as microbial contamination can occur, which may lead to permanent eye damage.
- Consult your doctor or eye care professional before using any eye medication with contact lenses.

CONTRA-INDICATIONS

- Do not use in case you are allergic to one of the ingredients.

PACKAGING

MeniCare Pure is available in 250mL and 70mL bottles, including a disposable lens case.

PRECAUTIONS

- Once opened, use *MeniCare Pure* within 3 months and make sure that the bottle is tightly closed after each use.
- Do not touch the bottle top with your hands.
- Do not swallow.
- Keep out of reach of children.
- Do not use after expiry date.
- Do not use if the film around the cap is broken before first use.
- Do not use with soft contact lenses.
- Keep the lens case clean and dry after use.
- Store at room temperature. (≤25°C)
- Store out of sunlight.
- Replace the lens case with every new bottle of *MeniCare Pure*.
- Consult your eye care professional before changing solutions to a different solution.
- Do not use as lubricating and rewetting solution.

ADVERSE REACTIONS

If you experience persistent discomfort or irritation, remove your lenses, stop using the product and consult with your eye care professional.

CONTENTS

- E-Polylysine 0,01% as preservative, Edetate disodium 0,05% as chelating agent, HCO-60 0,1% as surfactant.

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© 2016, Menicon Co., Ltd.

This instructions for use contains important information.
Please read carefully and retain for future reference.

PROGENT

EN Intensive cleaner

INDICATIONS

Protein remover and disinfecting solution for all rigid gas permeable lenses.

DIRECTIONS

- 1 Clean your lenses thoroughly with regular cleaning solution. Place the lenses into Progent Vial caps in its appropriate right or left holder : R = right, L = left, for intensive cleaning.
- 2 Open both doses carefully, by holding upright. Gently and slowly twist the tip, without touching the opening or squeezing the dose.
- 3 Insert the tip of dose inside the Progent Vial and fill slowly, first with A solution, then B solution, taking care not to spill any solution.
- 4 Close the Progent Vial cap securely gently shake it a few times.
- 5 Soak the lenses in Progent for 30 minutes maximum.
- 6 Remove the Progent Vial cap, empty its content. Rinse the lenses in the holder thoroughly with the regular rinsing solution.
- 7 Then clean the lenses thoroughly with the regular solution and rinse them. The lenses may then be worn or stored in the storage case.
- 8 Rinse the Progent Vial and its cap thoroughly with the rinsing and regular solution and let them air dry.

WARNINGS

Do not apply Progent to the eyes, the nose and the mouth. In case of misuse, rinse immediately your eyes thoroughly with water and consult your eye care practitioner.

- It is essential to rinse the lenses abundantly with a rinsing and regular solution after soaking them in Progent solution.
- KEEP OUT OF CHILDREN'S REACH.

PRECAUTIONS

- Do not soak lenses in Progent for over 30 minutes. Lenses might discolour. However this will not affect their quality.
- **Important : do not swallow.**
- To be used weekly for daily use (and on removing extended wear lenses)
- Do not use Progent A and B solutions separately.
- Single use doses. Discard the solution and the doses after each use. ☒
- Store at room temperature (15-25°C).
- Avoid contact with the skin and do not spill Progent on your clothes. It may bleach them.
- Do not use Progent A and B after expiry date printed on the single dose.
- Store out of direct sunlight.

CONTRA-INDICATIONS

- Do not use for soft lenses.
- Do not use if allergic to one of the ingredients.

ADVERSE REACTIONS

If you experience persistent discomfort or irritation, remove your lenses, stop using the product and consult your eye care professional.

CONTACT LENS DISINFECTION




Progent acts as a disinfectant for microorganisms (bacteria, moulds, viruses, amoebae) in just 5 minutes of soaking.

PACKAGING

Progent is available in boxes of 5 pairs of Progent A and B with one Progent Vial and starter kit.

COMPOSITION

Solution A : NaClO 20,3 mg, purified water ad 5 ml/dose.
Solution B : KBr 30,75 mg, purified water ad 5 ml/dose.

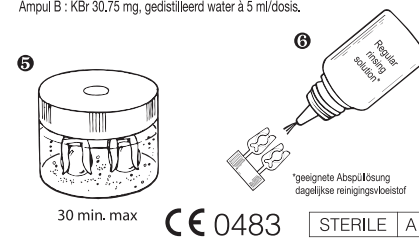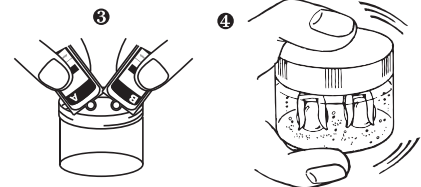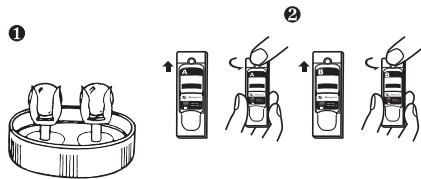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Menicon Pharma S.A.S. - Bd Sébastien Brant
Menicon SAS - 104 rue Marthe - F - 92583 Clichy Cedex
Menicon GmbH - Jakob-Latscha-Straße 3 - 60314 Frankfurt
Menicon Limited - Gatelodge Close Round Spinney - Northampton NN3 8 RJ.
© 2016, Menicon Pharma SAS